

# Operations checklist



Name: \_\_\_\_\_ Date: \_\_\_\_\_


## Battery operation

Unplugging the pump from the AC line will automatically put the pump on battery power. To recharge the battery, plug the pump into an AC wall outlet. The pump will automatically begin charging.

## General instructions


- Fill or spike feeding container and water bag prior to setting up pump.
- Press the **Power** button in the center to turn pump on.
- In order to achieve proper accuracy, the feeding set bag must be hung so that there are 10 inches (+/- 0.3) from the top of the pump to the top of the formula.
- Select **>Keep Settings** or **>Clear Settings** (for day-to-day use with the same feeding regimen, select **>Keep Settings** option).
- Load the set per the animation on screen.

## Setting a continuous feed


- Prime the pump if not completed already.
  - **Warning:** Patient must not be connected to Kangaroo OMNI Enteral Feeding Pump when priming is performed.
  - Select **>Prime**, then use the buttons on the left to either **>Auto Prime** or **>Manual Prime** the pump. If using manual prime, prime the flush tube before the feed tube when using feed sets with flush bags. If the feeding set is already primed, press the **>Skip Prime** button.
- Select **>Adjust Rate**, then use the buttons on the left to program the pump from 1 to 400 mL in increments of 1 mL when using a Feed Only or Feed/Flush feeding set. If you are using a thick formula feeding set, the pump can only be programmed from 1 to 200 mL in increments of 1 mL. Select **>OK** when desired rate is set.
- Select **>Start**. You will notice a small black drop  scrolling down the screen when feeding.

## Setting an intermittent feed

- Select **>Prime**, then use the buttons on the left to either **>Auto Prime** or **>Manual Prime** the pump. If using manual prime, prime the flush tube before the feed tube when using feed sets with flush bags. If the feeding set is already primed, press the **>Skip Prime** button.
  - **Warning:** Patient must not be connected to Kangaroo OMNI Enteral Feeding Pump when priming is performed.
- Press **>More Options**. Use the buttons on the left to navigate to Intermittent Mode. Press **>Select** to toggle between on and off. After turning intermittent mode on, press **>Done**.
- Select **>Adjust Feed**.
- Select **>Adjust Rate**. Use the buttons on the left to program the pump in increments of 1 mL. Select **>OK** when desired rate is set.
- Select **>Adjust # Feeds**. Use the buttons on the left to set the number of feeds (maximum of 99). Select **>OK** when desired number of feeds is set.
- Select **>Adjust Volume**. Use the buttons on the left to program the volume of the feed from 1 to 3000 mL. Select **>OK** when desired volume is set.

- Press **> Done**.
- Select **> Start**. You will notice a small black drop  scrolling down the screen when feeding.
- The **Feed Complete** notification will appear after completion of the programmed feeding.

### Setting the flush regimen

- Select **> Adjust Flush**, then **> Adjust Volume** to set the volume of water per flush cycle to be administered from 1 to 500 mL in increments of 1 mL. Select **> OK** when desired volume is set.
- Select **> Adjust Interval** to define the time interval between the start of each flushing cycle from one to 24 hours in one-hour increments. Select **> OK**. Select **> Done**.
- Select **> Start**. You will notice a small blue drop  scrolling down the screen when flushing.

### Re-priming the pump after the feed bag empties

- A feeding set bag (or container) that has been emptied will trigger the **Feed Error** screen. In this condition, refill the pump set bag or use a new feeding set bag (or container) to re-prime the set before continuing the feed.
- Disconnect the feeding line from the patient.
- Refill the bag or hang and spike the new feeding set bag (or container).
- Press **> Continue** on the feed error screen.
- Press **> Prime**, then press **> Auto Prime**.
- Press and hold **> Prime Flush** water exits down the downstream side of the cassette for a Feed Set with a Flush Bag.
- Press and hold **> Prime Feed** until formula has reached the end of the set.
- Press **> Done**, then select **> Start**.

For more information, contact your Cardinal Health sales representative, call customer service at **877.CARDINAL** or visit [cardinalhealth.com/kangaroo](https://cardinalhealth.com/kangaroo)

Prior to use, refer to the instructions for use supplied with this device for indications, suggested procedure, warnings and cautions.

© 2024 Cardinal Health. All Rights Reserved. CARDINAL HEALTH, the Cardinal Health LOGO, KANGAROO, Kangaroo OMNI and the Kangaroo LOGO are trademarks of Cardinal Health and may be registered in the US and/or in other countries. All other marks are the property of their respective owners. Lit. No. 2MS23-2518268-10 (02/2024)

